A medical professional in a white coat and green pants is examining a patient's leg. The patient is seated in a modern, brown leather Lasata patient recliner. A digital screen above the patient displays a video call with another person. The background shows a clinical setting with cabinets.

# Lasata patient recliner

Instructions

**carolina**  
an OFS company

[carolina.ofs.com](http://carolina.ofs.com)

800.763.0212

05.23

A medical professional in a white coat and green pants is examining a patient's knee while seated in a modern, reclining patient chair. The patient, a woman with blonde hair, is seated in the chair, facing the doctor. In the background, a monitor on a stand displays a video call with another person. A 3D anatomical model of a human leg is also visible on the screen.

# Lasata reclinable del paciente

Instrucciones

**carolina**  
an OFS company

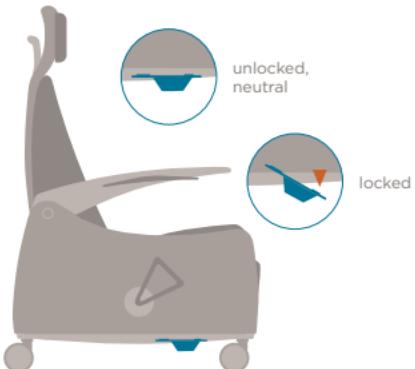
[carolina.ofs.com](http://carolina.ofs.com)

800.763.0212

05.23


### Dual sided central locking caster pedals

- With foot, apply pressure on the red lock symbol. By design, all four casters lock securing the recliner in a locked position.
- With foot, apply pressure on the green unlock symbol. The casters will unlock when in the neutral, level position of the pedal

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
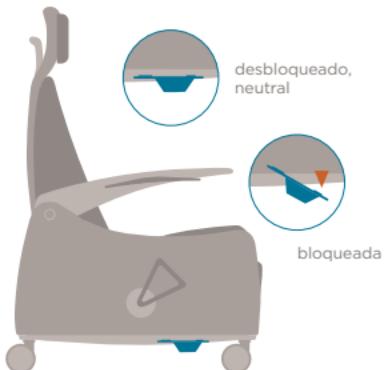
### **Foot ottoman handle**

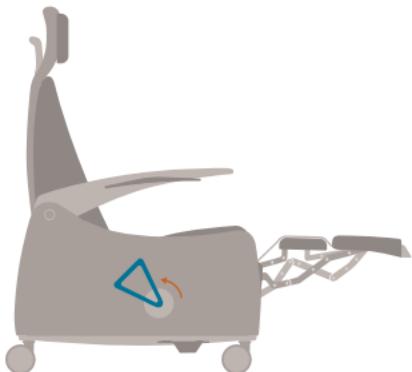- Using your hand, grip the foot ottoman handle and rotate from front to back of the recliner for foot ottoman to extend
- Using your hand, grip the foot ottoman handle and rotate from back to front of the recliner to close the foot ottoman

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
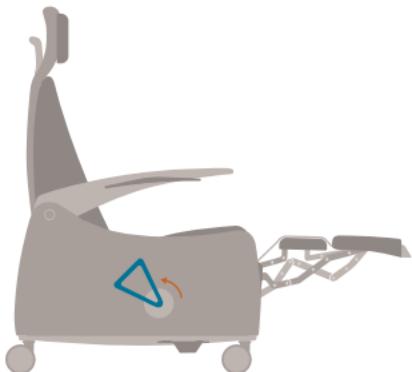
### Dual back recliner activation paddles

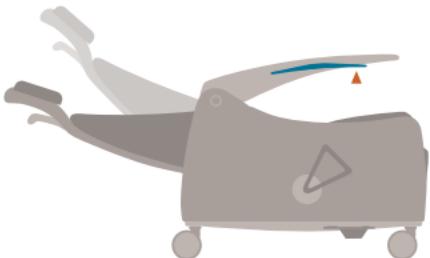- Located under each arm, press the paddles for back to activate and recline
- Located under each arm, press the paddles for back to activate to return to upright position

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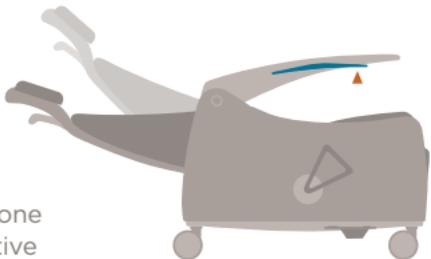
### Dual sided trendelenburg pedal

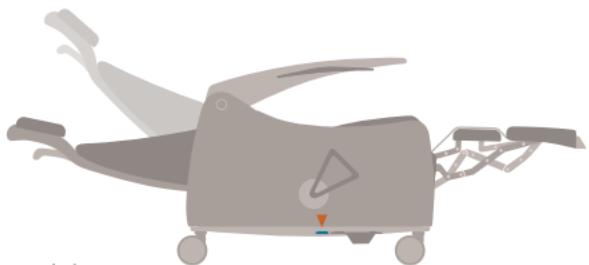- Begin with the unit fully reclined and the foot ottoman extended. To activate, step on trendelenburg foot pedal (heart) while applying pressure on the recliner's back to lock into Trendelenburg position.
- To return recliner to its normal reclined position press down on trendelenburg pedal and recliner automatically return to normal position

*Trendelenburg is not offered on the bariatric models*

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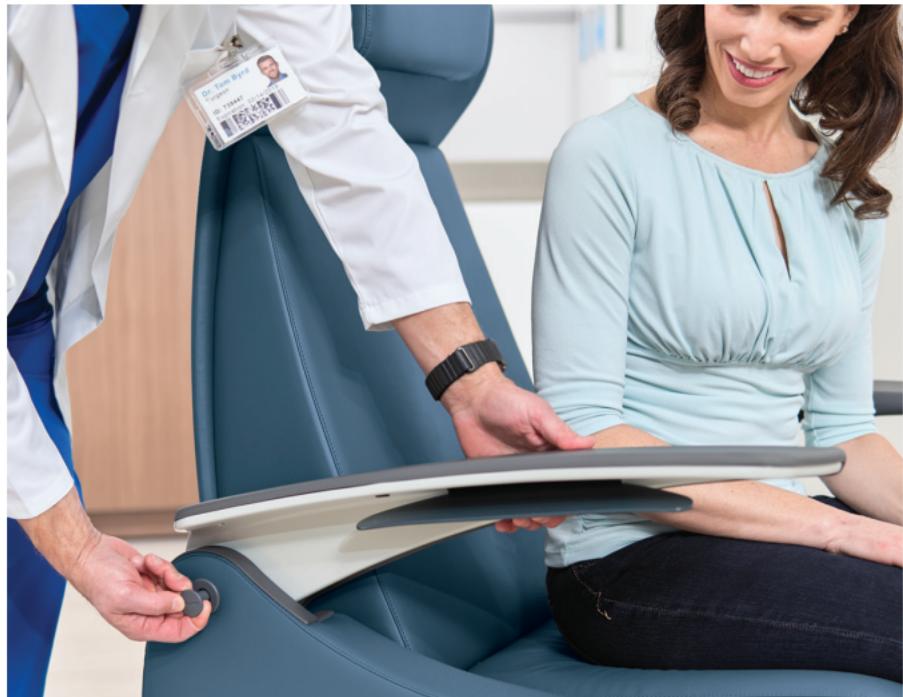
### Dual sided pivoting arm lock



- In the down position, the arm is completely locked
- For the arm to pivot up, simply push inward on the arm lock while lifting the pivoting arm upward. Release the arm lock once the arm is lifted
- Once placed in down position, the arm will automatically re-lock

*Dual-sided pivoting arms is a model in the Lasata series. All Lasata recliners are also offered without this feature. Dual-sided pivoting arms are not offered on bariatric models, they remain stationary.*

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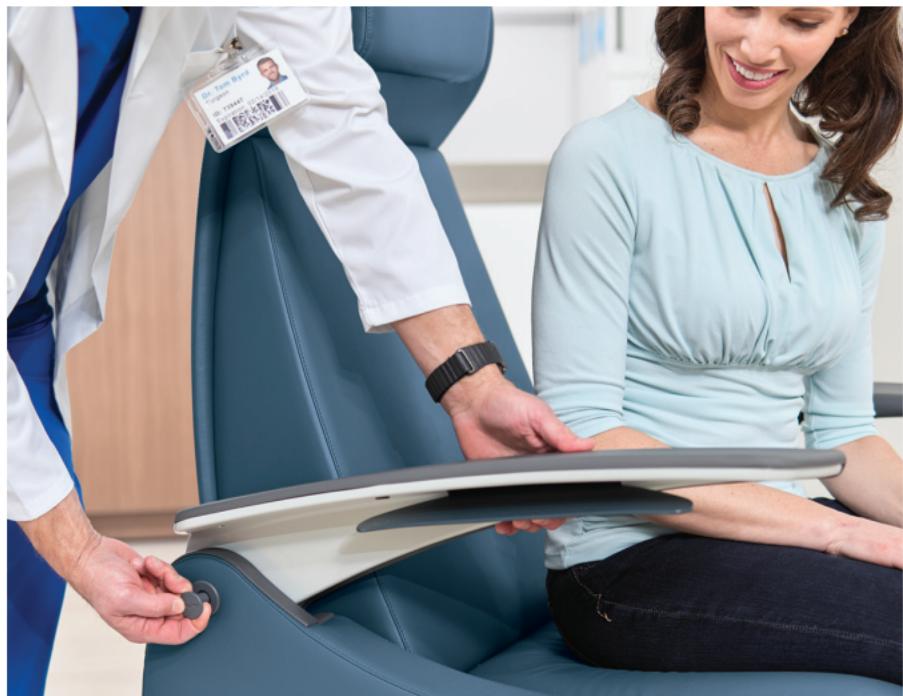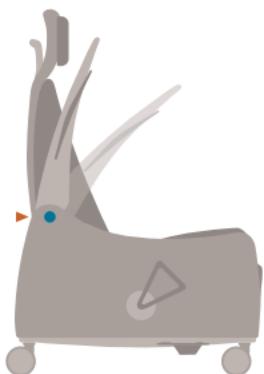## **Care and maintenance**

Carolina products are manufactured with careful attention to detail and with an on-going commitment to achieve a level of quality that is free of defects in materials and workmanship, given normal use and proper care.

Carolina offers an extensive program of graded-in textiles. Regular cleaning of the chair will enhance longevity of the product.

Activate chair mechanisms periodically to ensure proper function of moving parts.

## **Safety information**

- Do not sit on arms
- Do not sit on extended foot ottoman
- Close the foot ottoman when not in use
- Do not use recliner as patient transport
- Do not lift the chair by the outside back
- Keep casters locked when not in use
- Do not remove this tag

For more information refer to warranty, terms and conditions on our website.

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